

## Explanation of Symbols

### Symbol Indication

 This device complies with the requirements of the Medical Devices Regulation (2017/745/EU).

 Manufacturer


 Follow Instructions for Use


 Use By







 Serial number

 Medical Device

 Unique Device Identifier

 Importer

 Indicates the packaging material is Non-Corrugated Fiberboard (paperboard)

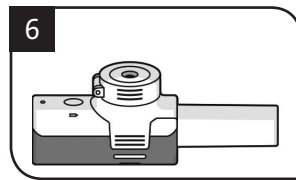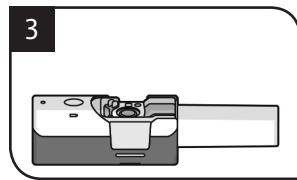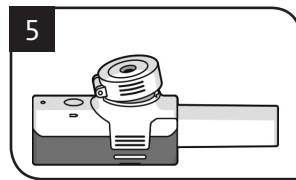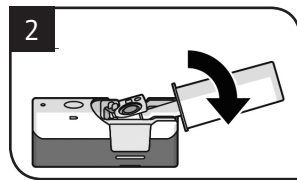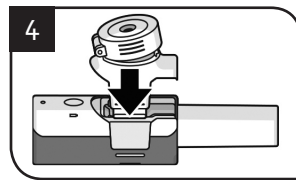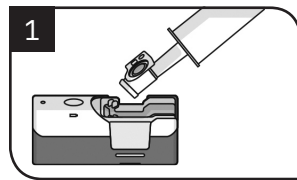

90360048

# Breelib™

## Monthly Pack Instructions for Use



**Vectura Group Ltd**  
One Prospect West, Chippenham, Wiltshire, SN14 6FH,  
United Kingdom,  
www.vectura.com



**Vectura Ireland Limited**  
Block 2, Newtown Business and Enterprise Centre  
Newtownmountkennedy  
Co. Wicklow, A63 XV26  
Ireland



 0123



DAW-00059-04  
500059R4  
280923



Packaging Technology Berlin sqwx page 1  
client: GVDE material-no.: 90360048  
Reference-Code: 500059R4 CMO-Code: DAW-00059-04 PZ: 2782A-2 code-no.:  
name: LF BREELIB MONTHLY CONSUMABLES IFU date: 280923  
colors: Black / CYAN / MAGENTA / YELLOW country: GB/-

dimension: 255 x 85 mm  
Restricted Document

## 1. Safety Information

### Warning

- ▶ The Breelib™ Monthly Pack is an accessory to the Breelib™ Inhalation System. Read the entire Instructions for Use of the Breelib™ Inhalation System before using the device. Correct and safe operation can only be achieved if Instructions for Use of the Breelib™ Inhalation System are followed.

- ▶ Do not use the Breelib™ if it is damaged or modified. Only use accessories supplied by an authorised service partner.
- ▶ Only Vectura Group Ltd or a service partner authorised by Vectura Group Ltd is permitted to repair the Breelib™. Vectura Group Ltd is not liable for any damage or malfunction as a result of unauthorised repair.

- ▶ Do not share your Breelib™ with other people as it can lead to infections. The Breelib™ is designed for single patient use.
- ▶ Ensure the Breelib™ has been properly cleaned before use according to the Instructions for Use of the Breelib™ Inhalation System.

### Caution

- ▶ Do not use the components of the Monthly Pack for more than one month to maintain constant performance.

## 2. Package Content

- ▶ Instructions for Use (this document)
- ▶ Mouthpiece
- ▶ Nebuliser unit

## 3. Intended Use

The Breelib™ Monthly Pack is an accessory to the Breelib™ Inhalation System. The Breelib™ Monthly Pack can only be used in combination with the Breelib™ Inhalation System. For information regarding intended use, general instructions and the description of the device refer to the Instructions for

Use of your Breelib™ Inhalation System.

## 4. Storage and Transportation

No specific storage and transport conditions are defined for the Breelib™ Monthly Pack.

## 5. Service life

The components of the Breelib™ Monthly Pack are intended for repeated use. The service life of the Breelib™ Monthly Pack is limited to one month.

## 6. Disposal

Components of the Monthly Pack may be disposed of with normal household waste.

**Bayer**  
90360048